





# Assembly Instructions

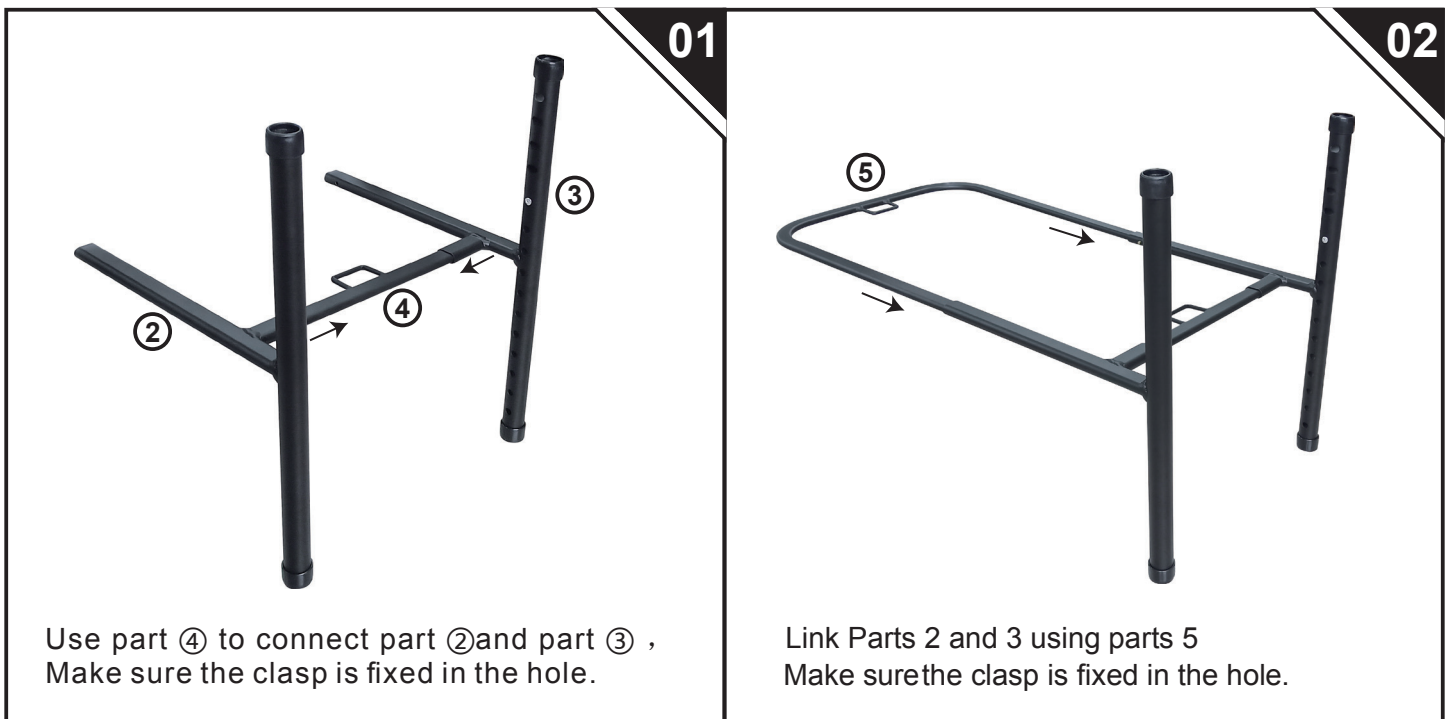

03

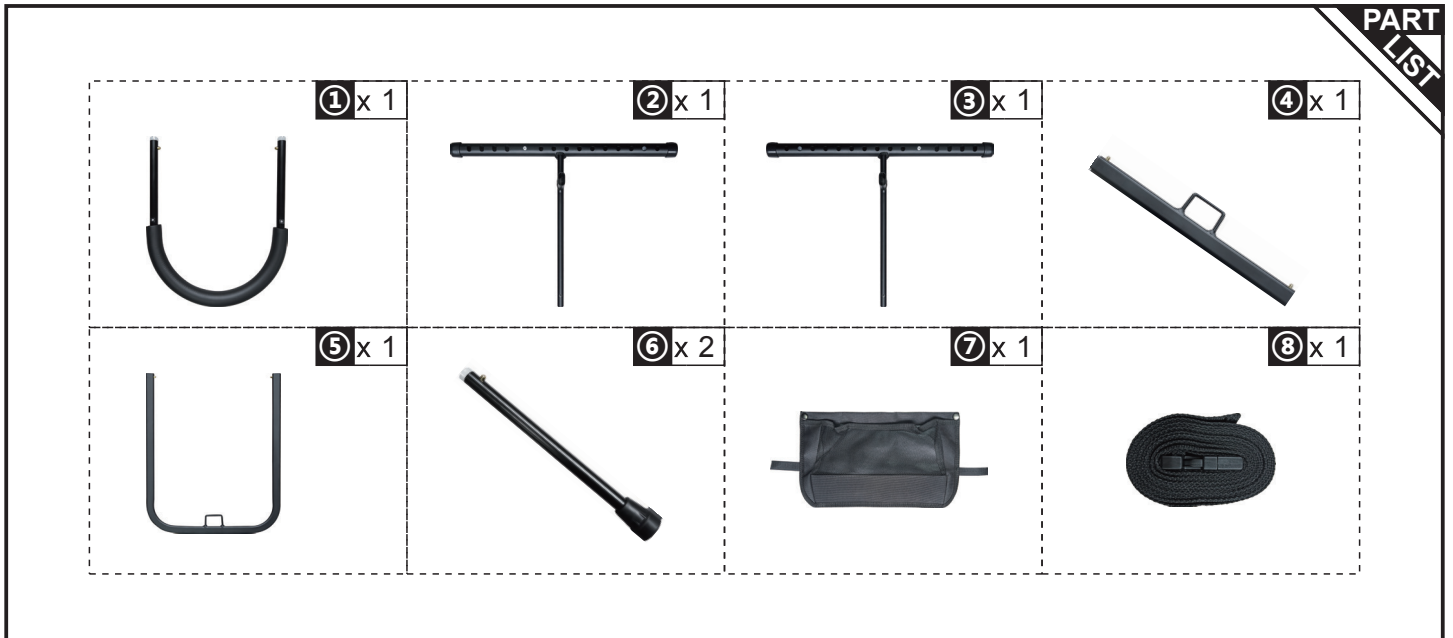

Connect the pipes with fitting 2, taking care to adjust the proper height and make sure the two pipes are on the same horizontal line.

04



Install the handrails and make sure that the snap has lodged in the pipe and that the proper height has been set.

05



Hang the bag and attach the velcro to the left and right tubes (Pack no more than 5 kg)

06



Place the tube under the mattress and wrap the rope around the mattress. Secure and fasten the rope to the mattress.



### WARNING

1. Please check whether the fittings are complete before installation
2. Before use, make sure that the fasteners are properly installed and tightened, and that the clips are at the same height
3. When using, be sure to use in a smooth situation
4. Please pay attention to safety when using, if necessary, please arrange nursing staff to assist use
5. For a period of time each use, please carefully check whether the fastener is loose, the wear of the pad, if necessary, please replace in time
6. People with limited mobility or older people should be accompanied by others to prevent secondary injury caused by improper use